

Adapt Lubricating Deodorant

Ostomy Care Tips

General Information

- Adapt Lubricating Deodorant destroys odors, eases emptying and helps prevent pouch static and sticking.
- It is not a perfume or masking agent.
- Adapt Lubricating Deodorant should be used in drainable and closed pouching systems only.



Figure 1

For Drainable Pouches

1. Apply pouch.
2. **Bottle:** add approximately 5ml or 1 tsp. (Figure 1).
Packet: add the complete contents of the packet. (Figure 2)
Close the pouch in the normal manner.
3. Rub to thoroughly coat the inside of the pouch (Figure 3).
Repeat Steps 1 and 2 after every emptying.



Figure 2

For Closed Pouches

1. Do not remove release paper on barrier to add product.
2. **Bottle:** add approximately 5ml or 1 tsp. (Figure 4).
Packet: add the complete contents of the packet. Apply your pouch.
3. Rub to thoroughly coat the inside of the pouch (Figure 3).



Figure 3

Frequently Asked Questions

Q: Does it matter if the Adapt Lubricating Deodorant comes in contact with the filter on the pouch?

A: When coating the inside of the pouch, it is best to avoid the filter.

Q: Is Adapt Lubricating Deodorant covered by Medicare?

A: Yes, Adapt Lubricating Deodorant is billed per ounce using code A4394 (Ostomy deodorant for use in ostomy pouch, liquid). Medicare has not published a usual maximum quantity for this code. Both the 8 oz. bottle and 8 ml packets are billed using the same code. When billing the bottle, 8 units of A4394 will be billed. When using the packets (50 per box), 13 units of A4394 will be billed.

Q: Do I need to use the Adapt Lubricating Deodorant every time I empty my pouch?

A: Yes. For the product to work effectively, it must be reinserted each time your pouch is drained.



Figure 4



Frequently Asked Questions

Q: Can I just put a few drops in the bottom of my pouch and not rub it against the sides of the pouch?

A: No, this is not recommended since it will not be effective as a pouch lubricant or as a pouch deodorant. Rubbing the pouch helps to coat the inside of the pouch film.

Q: Wouldn't it be easier and cheaper to use a cooking spray instead to coat the inside of my pouch?

A: This is not recommended because cooking oil may cause the seal of your pouch to fail. Also, cooking oil sprays do not eliminate odor. In addition, Adapt Lubricating Deodorant is reimbursed by many insurance companies as a pouch deodorant which makes it more cost effective to use than cooking oil.

Q: The 8 ounce bottle is not convenient to take into a public restroom. Is there a smaller bottle available?

A: Adapt Lubricating Deodorant is also available in single-use packets. This is much more discrete and convenient when you are away from home.

Q: It doesn't have a scent. How do I know it is working?

A: Adapt Lubricating Deodorant is not a perfume or a fragrance used to mask or cover-up odors. It does not have any odor by itself, but it will trap odors in the pouch.

Q: Can the Adapt Lubricating Deodorant come in contact with the stoma?

A: Yes, it has been tested and it will not harm your stoma.

Q: What if Adapt Lubricating Deodorant comes in contact with my skin?

A: This product is not a skin irritant. It is water soluble so it will easily be removed from the skin with a moist cloth.

Note: Avoid contact with the adhesive on the skin barrier or flange.
For use in ostomy pouches only.



Adapt Lubricating
Deodorant
78500/78501



Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA
1.800.323.4060

www.hollister.com