

AQUACEL® Hydrofiber® Wound Dressing



AQUACEL Hydrofiber Wound Dressing is a soft, sterile, nonwoven pad or ribbon dressing composed of sodium carboxymethylcellulose. This dressing may be premoistened with saline for dry wounds.

AQUACEL Hydrofiber Wound Dressing is indicated for:

—Management of exuding wounds including leg ulcers, pressure ulcers and diabetic ulcers, surgical wounds, partial thickness burns, and traumatic wounds*

Why choose AQUACEL Hydrofiber Wound Dressing?


- Absorbs and retains exudate and harmful components
- Locks exudate in the dressing
- Conforms to the wound surface, thereby reducing dead space
- Does not damage tender, granulating wound tissue or healthy tissue surrounding exuding wounds during dressing changes
- Absorbs wound fluid and creates a soft gel
- Aids in autolytic debridement (removal of nonviable tissue from the wound)
- Removes without leaving residue or causing trauma to wound
- Manages painful wounds

*Under the supervision of a healthcare professional.




Directions for using AQUACEL® Hydrofiber® Wound Dressing


Before dressing the wound, make sure to clean, examine, and assess the area. Debride if necessary.

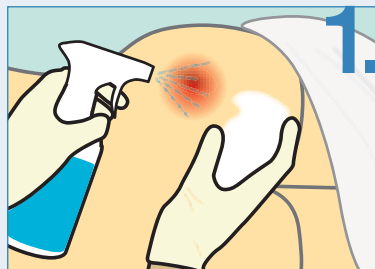

1. Use a cleansing solution on the wound surface and the surrounding skin, and dry the skin thoroughly.

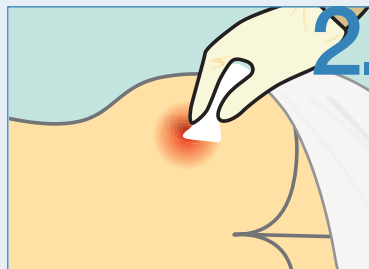

2. Loosely pack ribbon or pad into deep wounds to about 80% of the depth of the wound to accommodate swelling of the dressing.

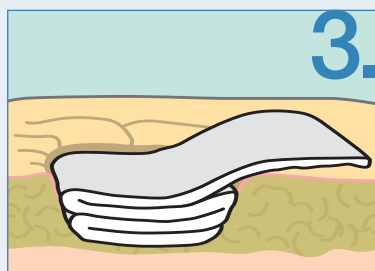

3. Leave a small overhang (at least 2.5 cm or 1 inch) to facilitate removal. Excess dressing can be cut to size without shedding fibers.



4. Cover the ribbon or pad with a moisture-retentive dressing. Secure the dressing with hypoallergenic tape, if necessary.

Dressing can remain in place for seven days maximum

—Change the dressing when clinically indicated or when the softened area approaches the edge of the dressing

Removal of the dressing

1. Grasp one corner of the dressing and carefully lift away from wound
2. In the unlikely event of adhesion to the wound bed, the dressing can be easily removed by soaking

Ordering information

Catalog #	Size	Quantity	HCPCS
177901	2" x 2"	10 per box	A6196
177902	4" x 4"	10 per box	A6196
177903	6" x 6"	10 per box	A6197
177904	3/4" x 18" (Ribbon)	5 per box	A6199

Examples of products that may be used with AQUACEL Hydrofiber Wound Dressing

SÄF-Cleans® AF Dermal Wound Cleanser

An advanced surfactant-containing formula that washes away debris that saline may leave behind. A no-rinse formula with an adjustable nozzle.

DuoDERM Signal® Dressing

A thin, flexible, conformable, and moisture-resistant wound dressing that covers susceptible areas anywhere on the body to gently protect newly formed tissue or skin at risk of breakdown. Indicates with dotted line when it is time to change the dressing.

Versiva® XC™ Dressing

A thin, flexible, conformable, and moisture-retentive wound dressing that combines the gelling action of Hydrofiber technology with foam to protect periwound skin from the risk of maceration.

Please see package insert for complete Instructions for Use.

The reimbursement information provided by ConvaTec is intended to provide general information relevant to coding and reimbursement of ConvaTec products only. Coverage and payment policies for the same insurer can vary from one region to another and may change from time to time because of ongoing changes in government and insurance industry rules and regulations. Therefore, please confirm HCPCS codes with your local DME-MAC, private insurer, or Medicaid agency before processing claims. ConvaTec does not guarantee coverage or payment of its products listed herein.

To learn more, ask your representative or call
1-800-422-8811
M-Th, 8:30AM - 8:00PM, ET
Fri, 8:30AM - 6:00PM, ET
Sat, 10:00AM - 2:00PM, ET
www.convatec.com