
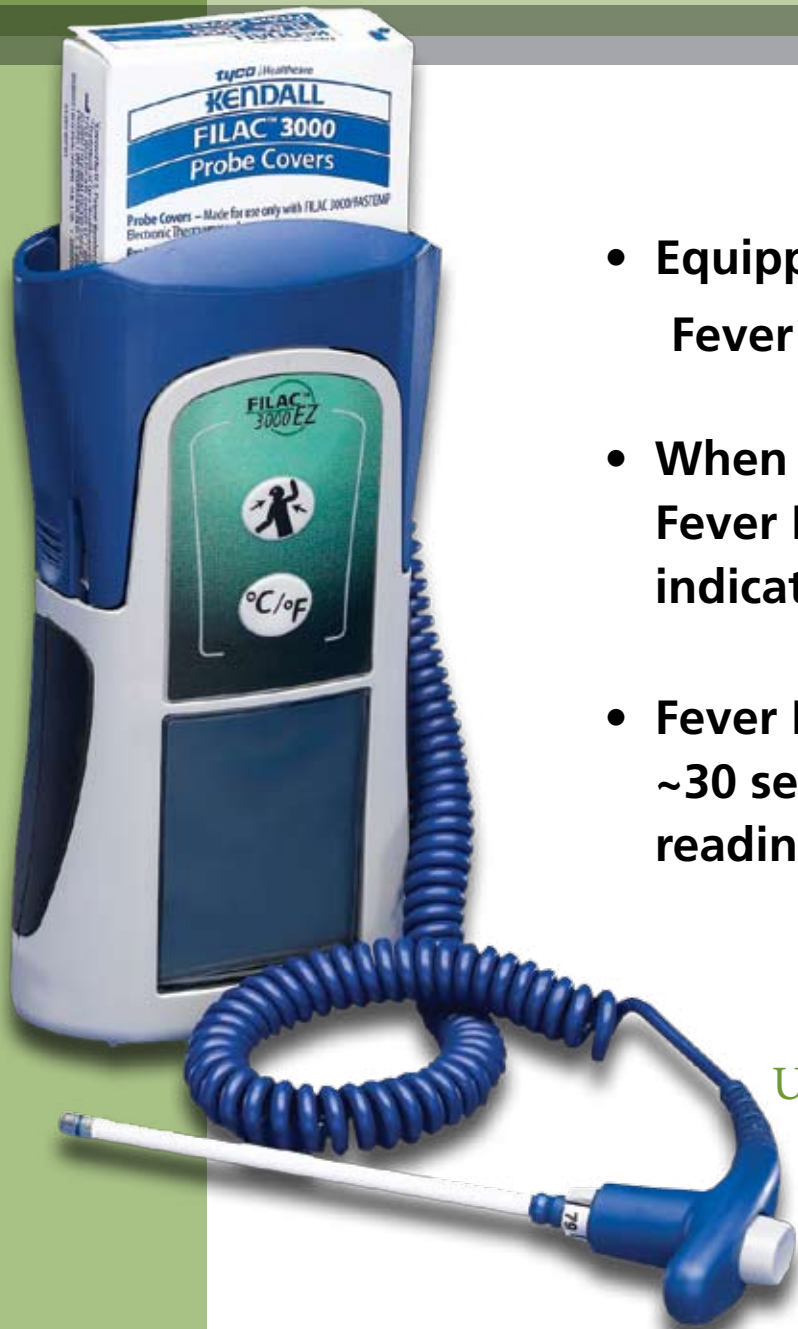


Filac™ 3000 AD and Filac™ 3000 EZ

Electronic Thermometer

Fever Mode Reference Guide

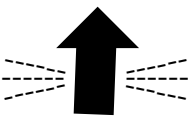

- Equipped with an advanced feature called **Fever Mode**
- When a temperature of 99.7°F is sensed **Fever Mode** begins automatically and is indicated by a **blinking** "up arrow" icon 
- **Fever Mode** extends measurement time by ~30 seconds to provide the most accurate reading

ALWAYS HOLD PROBE IN PLACE
UNTIL MEASUREMENT IS COMPLETE

	TEMPERATURE SITE			
	0-2 Years	3-10 Years	11-65 Years	>65 Years
Ear	97.5° - 100.4°F 36.4 - 38°C	97.0° - 100.0°F 36.1 - 37.8°C	96.6° - 99.7°F 35.9 - 37.6°C	96.4° - 99.5°F 35.8 - 37.5°C
Oral		95.9° - 99.5°F 35.3 - 37.5°C	97.6° - 99.6°F 36.7 - 37.6°C	96.4° - 98.5°F 35.7 - 36.9°C
Core	97.5° - 100.0°F 36.4 - 37.8°C	97.5° - 100.0°F 36.4 - 37.8°C	98.2° - 100.2°F 36.8 - 37.9°C	96.6° - 98.8°F 35.9 - 37.1°C
Rectal	97.9° - 100.4°F 36.6 - 38°C	97.9° - 100.4°F 36.6 - 38°C	98.6° - 100.6°F 37 - 38.1°C	97.1° - 99.2°F 36.2 - 37.3°C
Axillary	94.5° - 99.1°F 34.7 - 37.3°C	96.6° - 98.0°F 35.9 - 36.7°C	95.3° - 98.4°F 35.2 - 36.9°C	96.0° - 97.4°F 35.5 - 36.3°C

For more information call 800.962.9888
or visit us at www.covidien.com

COVIDIEN, COVIDIEN with logo and ™ marked brands are trademarks of Covidien AG.
©2009 Covidien AG. All rights reserved.

H6395 25M 01/09