

### SAVE THESE INSTRUCTIONS

*NOTE: Check all parts for shipping damage. In case of damage, DO NOT use. Contact Invacare for further instructions.*

### SAFETY SUMMARY

The following recommendations are made for the safe and proper installation and use of the Bed Buggy:

#### GENERAL WARNINGS

**DO NOT** install this equipment without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions and Instructions, contact a healthcare professional, dealer or technical personnel if applicable, before attempting to install this equipment - otherwise, injury or damage may occur.

#### INSTALLATION WARNINGS

Make certain **ALL** screws and nuts are tight at all times.

The Bed Buggy has a recommended weight limit of 100 lbs.

*Kit includes the following:*

ITEM NO.	PART NO.	QTY.	DESCRIPTION
1	53000M536	2	10-inch Wheels
2	53000M537	2	Hitch Pin
3	7000M352	1	Bed Buggy Weldment
4	53000M535	1	Instruction Sheet

### INSTALLING OF THE BED BUGGY


*NOTE: The following instructions apply to Manual or Electric Beds.*

1. Remove head and footboard assemblies. Refer to Owner's Manual P/N 51000M493 for Head/Footboard removal.
2. Assemble wheels onto 0.50-inch diameter tubing on each end of the Bed Buggy Weldment (FIGURE 1).

*NOTE: Make sure that the six (6) ribs of wheel are towards the inside of the axle assembly and the three (3) ribs are towards the outside (FIGURE 1).*

*NOTE: Slight tapping of wheel onto tubing may be required. Use a rubber or plastic mallet.*


3. Insert hitch pin through hole on 0.5-inch tubing of the Bed Buggy Weldment to secure the 10-inch wheels in place (FIGURE 1).

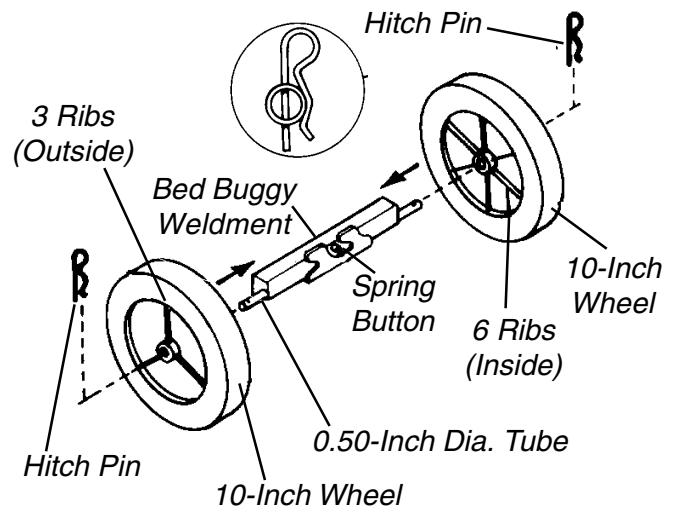

**FIGURE 1 - ASSEMBLING WHEELS**

4. Place bed buggy over bed spring hook (FIGURE 2).
5. Push down so that rivet heads of the bed spring hook depresses spring button and slide the bed buggy assembly to the left.
6. This will secure the bed buggy to the headspring or footspring of the bed assembly.
7. Rotate head/foot spring so that wheels are on the floor and roll unit to destination.
8. To remove bed buggy, depress spring button with index finger, slide the bed buggy assembly to the right and remove from bed spring hook.

*NOTE: The bed buggy can also be used for the head spring using the same attachment method.*

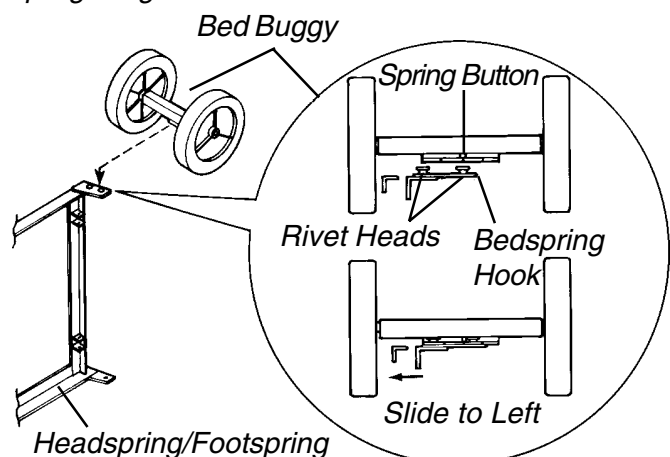

**FIGURE 2 - POSITIONING THE BED BUGGY ON THE BED ASSEMBLY**



Yes, you can!

Invacare Corporation

www.invacare.com

**USA**  
2101 East Lake Mary Blvd.  
Sanford, Florida 32773  
(407) 321-5630  
800-333-6900

**Canada**  
5970 Chedworth Way  
Mississauga, Ontario  
L5R 3T9, Canada  
905-890-8838

Invacare and "Yes, you can" are trademarks of Invacare Corporation.  
© 2000 Invacare Corporation  
Form No. 93-17 P/N 53000M535 Rev A (1) 3/00