


InterDry™ Ag

Textile with Antimicrobial Silver Complex

Patent Pending

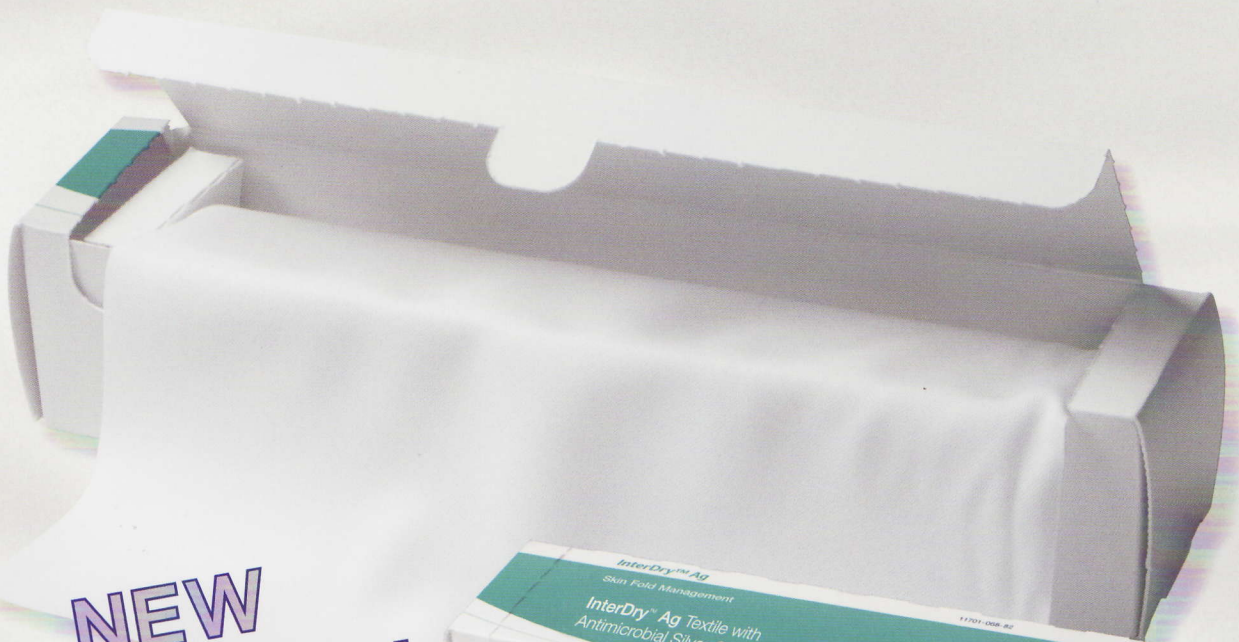

**NEW
INDICATION**



**The First Real Solution Designed
For Skin Fold Management ...**



Coloplast

InterDry™ Ag

NEW
INDICATION

Textile with Antimicrobial Silver Complex



Day 1



Day 5

Example of a patient with candidial intertrigo behind the knee after 5 days using **InterDry Ag** Textile with Antimicrobial Silver Complex. The patient wears lymphedema wraps.

THE CHALLENGE OF SKIN FOLD MANAGEMENT

Intertrigo - inflammation produced by chafing adjacent areas of skin


- Excessive moisture and friction leading to erythema, maceration and denudement
- Pain associated with itching and burning
- Frequently complicated by fungal and bacterial infections

A VERSATILE DEFENSE FOR HARD-TO-TREAT INTERTRIGO

- Silver impregnated formulation in the textile provides effective antimicrobial action for up to five days, reducing *Staphylococcus aureus*, *Staphylococcus epidermis*, *Pseudomonas aeruginosa* and *Candida albicans*
- Reduces or eliminates odor
- Polyurethane coating wicks moisture
- Specially designed knitted polyester textile reduces skin-to-skin friction
- Multi-use roll can be easily cut and applied to fit individual body locations

THE BEST OPTION

- In comparative *in-vitro* testing, **InterDry Ag** Textile with Antimicrobial Silver Complex outperformed commonly used products with superior wicking, translocation and evaporation of moisture, tensile strength, and the reduction of friction.
- 100% of symptoms associated with intertrigo were improved or resolved during five day clinical studies.

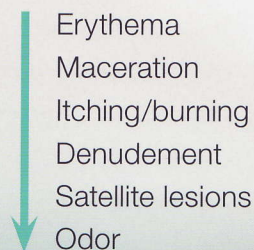


- **InterDry Ag** Textile is the only option for skin fold management containing an ionic silver complex for odor control.
- Effective for up to five days of continuous use.

ANSWERING THE SKIN FOLD MANAGEMENT CHALLENGE

How **InterDry™ Ag** Textile with Antimicrobial Silver Complex Works

This first of its kind efficient Skin Fold Management textile is specially designed to manage moisture, odor, and inflammation in skin folds and other skin-to-skin contact areas.

- Knitted 100% polyester textile provides a soft and low friction surface, minimizing the risk of skin breakdown
- Reduces microbial colonization in the fabric when placed within skin-folds
- A proprietary polyurethane coating allows for moisture translocation by wicking moisture within the fabric and efficiently evaporating moisture out of the skin fold – keeping skin dry
- Ionic silver is impregnated into the cloth providing continuous and effective antimicrobial odor control

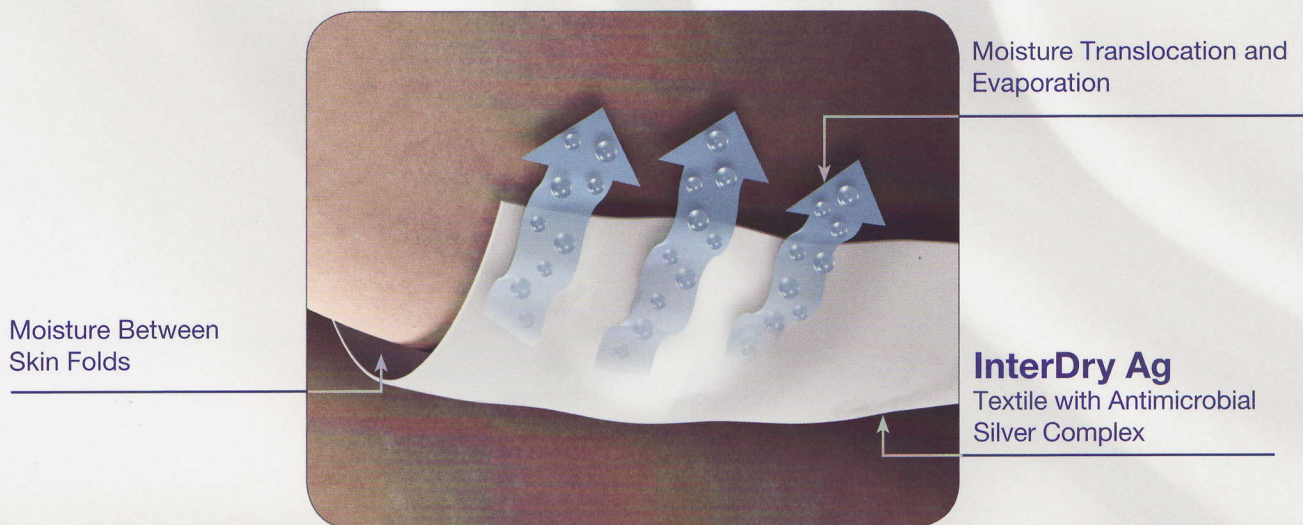

InterDry Ag is not a wound dressing, nor may it be used as a barrier against urine or feces. Federal law restricts this device to sale by or on the order of a licensed health care professional.

InterDry™ Ag Textile with Antimicrobial Silver Complex		
Product Code	Roll/Box	Case
7910	10 in. x 12 ft.	10 boxes

Patent Pending



Coloplast Corp.
Minneapolis, MN 55411
800.533.0464
usmedweb@coloplast.com
www.us.coloplast.com

Coloplast® is a registered trademark of Coloplast A/S or related companies.
InterDry™ is a trademark of Coloplast A/S or related companies.
© 2007-06 All rights reserved. Coloplast Corp., Minneapolis, MN USA.