

FIGURE 3 - RAISING/LOWERING STAND UP LIFT

MOUNTING BATTERY CHARGER (FIGURE 4)

NOTE: Refer to your local regulations concerning proper mounting procedures.

1. Place the battery charger with mounting bracket on the wall at the desired position.
2. With a pencil, mark the MIDDLE hole position.
3. Measure down 6-1/2-inches (16.5 cm) from the pencil mark and drill one (1) mounting hole.
4. Install the BOTTOM mounting screw until there is an approximate 1/8-inch (3 mm) gap between the screw head and the wall.
5. Install the battery charger with mounting bracket onto the bottom mounting screw.
6. Drill the remaining two (2) mounting holes.

7. Install the two (2) remaining mounting screws through the mounting bracket and into the wall. Tighten securely.
8. Plug the battery charger into the wall electrical outlet.

NOTE: ON LED should illuminate.

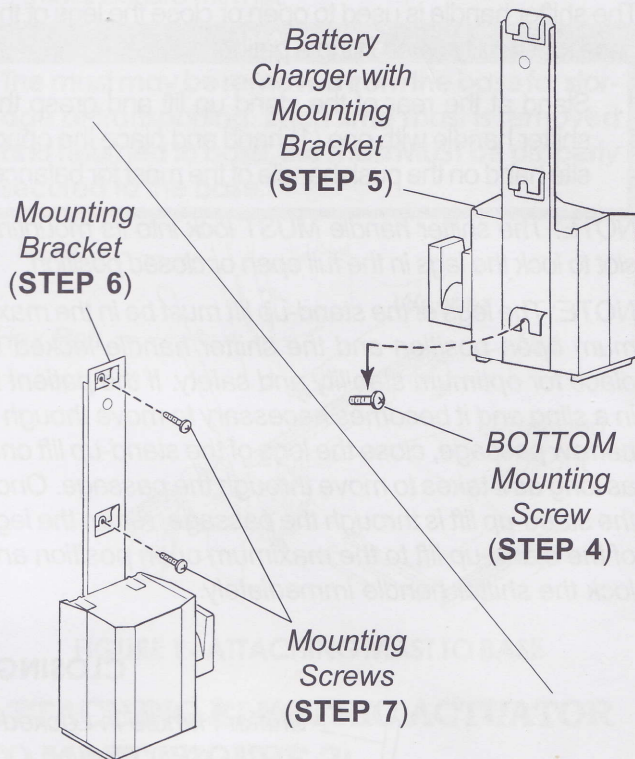


FIGURE 4 - MOUNTING BATTERY CHARGER

CHARGING BATTERIES (FIGURE 5)

NOTE: Invacare recommends the battery be recharged daily.

NOTE: An audible alarm will sound (horn will beep) when battery is low.

1. Lift UP on the handle on the top of the battery.
2. Lift the battery UP and OUT away from the control box.

WARNING

Make sure there is an audible click when mounting battery on the battery charger to confirm proper mounting. Otherwise, injury or damage may occur. Refer to FIGURE 5 for correct mounting.

3. Place the battery on the battery charger as shown in FIGURE 5. Make sure there is an audible click.

NOTE: The charge LED will illuminate. When charging is complete, the charge LED will stop illuminating.

NOTE: A battery needing to be fully recharged will take approximately four (4) hours.

4. Lift UP on the handle on the top of the battery.
5. Lift the battery UP and OUT away from the battery charger.

WARNING

Make sure there is an audible click when mounting battery on the control box to confirm proper mounting. Otherwise, injury or damage may occur. Refer to FIGURE 5 for correct mounting.

6. Reinstall the battery onto the control box as shown in FIGURE 5. Make sure there is an audible click.

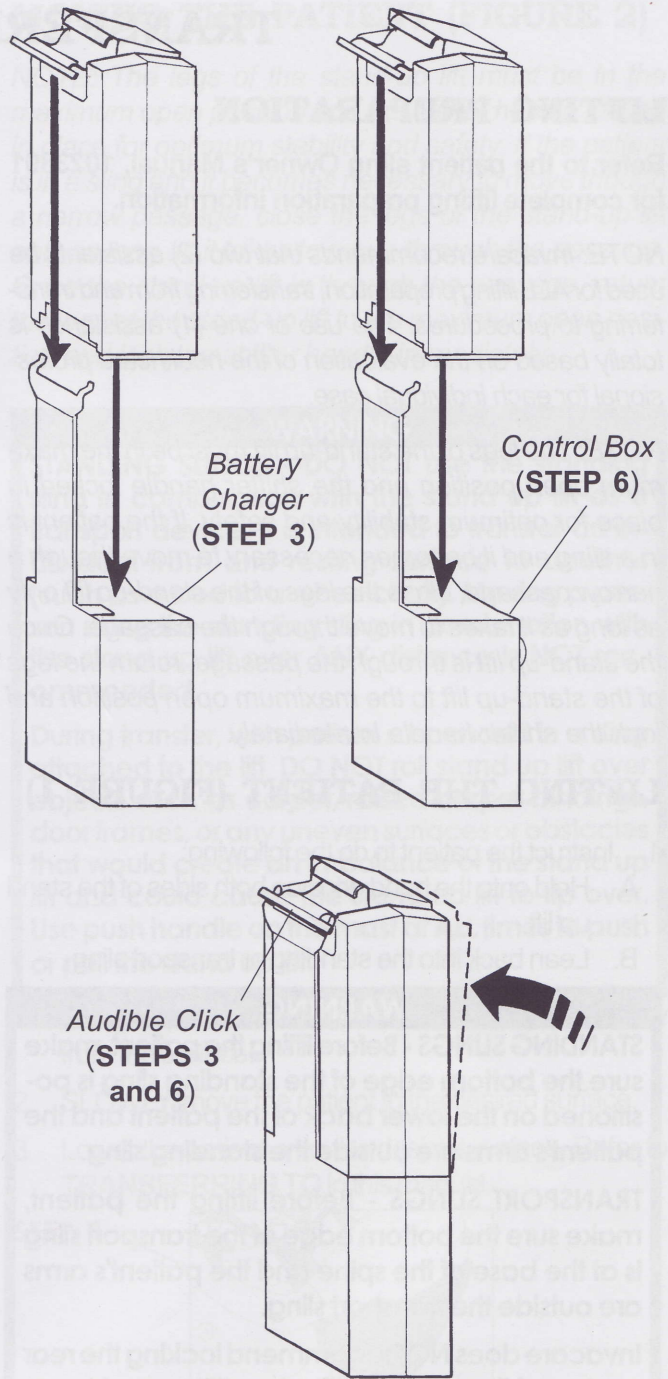
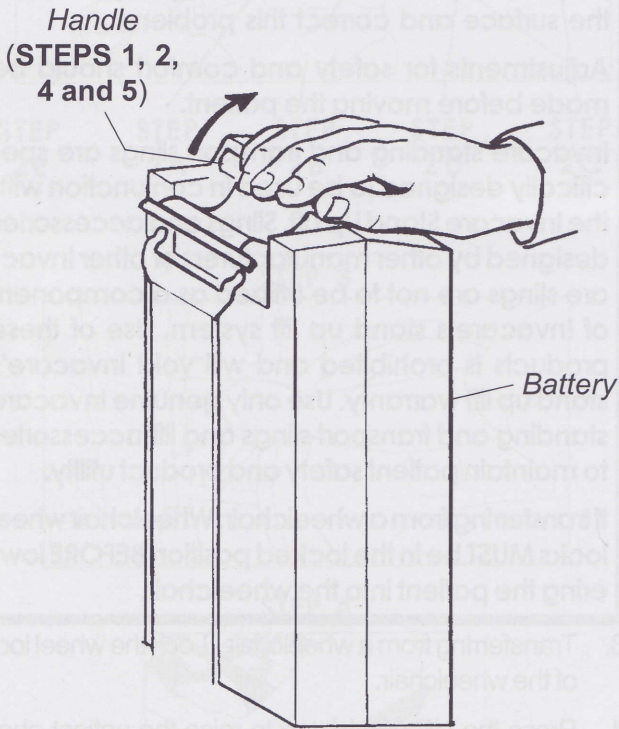


FIGURE 5 - CHARGING BATTERIES