

RediScreen™

Fecal Occult Blood Test Devices

6 GREAT FOBT PRODUCTS!

Available in Cassette or Guaiac Slide format.

Rapid, convenient, and qualitative methods for asymptomatic occult bleeding. Devices available in cassette format or guaiac slide—with both color enhanced sensitivity and readability.



Fecal Occult Blood Test Cassette—Immunoassay
20 Collection Tubes, 20 Test Cassettes, 20 per box
(Part Number: 2006)



Office Pack—Guaiac Slides
34 Triple Slide Tests, 2x10 mL Developers, Applicators, 34 per box
(Part Number: 2084)



Patient Pack—Guaiac Slides
100 Triple Slide Tests, 6x10mL Developers, Applicators, Foil Lined U.S. Postal approved Envelopes, 2x50 per box
(Part Number: 2080)



Lab Pack—Guaiac Slides
100 Single Slides Tests, 2x10mL Developers, Applicators, 100 per box
(Part Number: 2081)



Developer Solution—for Guaiac Slides
8x10 mL Developers, 8 per box
(Part Number: 2082)



Test Tape—for Guaiac Slides
2x100 Test Tape, 2x15mL Developers, 200 per box
(Part Number: 2083)

RediScreen™ Fecal Occult Blood Tests

Two variations of RediScreen™ fecal occult blood tests are available from the name you can count on. Tests include the immunoassay cassette, or the all new, RediScreen™-Ultra guaiac slide tests—formulated with ultra sensitive color development for enhanced readability.

Each device aids in the early detection of asymptomatic fecal occult bleeding.

Ideal Use

- Routine office practice
- Routine laboratory and hospital testing
- Mass screening programs

Features and Benefits

- Available **exclusively** through CIDA, StarLine and ABCO members
- 100% satisfaction guarantee
- Offered in several convenient and easy to use formats
- On board positive and negative monitors

RediScreen Immunoassay Test

- 98% accuracy and specificity
- No dietary restrictions
- Easy to use cassette with results in minutes
- Includes higher analytical sensitivity than standard guaiac-based tests


RediScreen-Ultra Slide Test

- Replaces former Private Label Fecal Occult Blood test kits from: CIDA, StarLine and ABCO
- Includes enhanced color sensitivity and readability, 20-30% less expensive than other FOBT brands

Procedure*—RediScreen™ Fecal Occult Test Cassette—Immunoassay


Unscrew collection tube cap and use sampler to pierce specimen in 5 different locations.

Insert sampler back into collection tube, tighten cap firmly

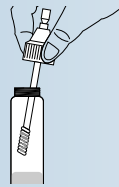

Shake collection tube to mix specimen.

Snap off tip of collection tube and dispense four drops of extraction Solution into sample well.

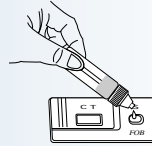

Read results within 5-10 minutes after adding extraction solution



Positive




Negative

Procedure*—RediScreen™-Ultra Fecal Occult Blood Test—Guaiac Slides


Collect stool sample.

On slide front, apply thin smear of stool inside Oval I and with different portion of stool apply smear to Oval II.

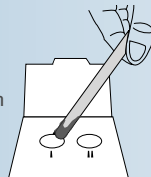


Slide Front

On slide back, apply two or more drops of RediScreen-Ultra Developer Solution to exposed paper.

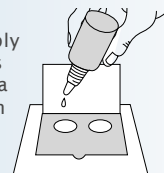

Slide Back

Read results between 30-60 seconds. Any trace of blue is positive. No indication is negative.

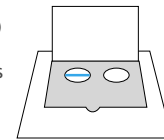

Positive

* See product insert for full details on instructions, limitations, and warnings.

Part Number Cross Reference

	RediScreen™ PerMaxim RediScreen™ - Ultra P/N	abco FOBT P/N (discontinued)	CIDA FOBT P/N (discontinued)	StarLine FOBT P/N (discontinued)	Hemocult® (Beckman Coulter®) P/N	Seracult® (Propper®) P/N	Colo- screen® (Helena Labs®) P/N	Hemascreen® (Immunos- tics®) P/N
100 Triple Slides - Patient Pack	2080	661100/ 664130	46-5070/ 46-5075	175071/ 175085	61130/ 64130	372007/ 375007	5071/ 5085	HSPP-50
100 Single Slides - Lab Pack	2081	660151/ 664151	46-5080/ 46-5085	175072/ 175086	60151/ 64151	371001- 374001	5072/ 5086	HS-100
Developer/ 8 Bottles	2082	N/A	N/A	N/A	N/A	N/A	N/A	HSDV-8
Test Tape - 2 x 100 Tests	2083	663202	46-5079	175079	63202	379010	5079	HSTK-2
34 Triple Slides - Office Pack	2084	661200	46-5082	175082	61200	372004/ 374002	5082/ 5087	HS-34

FOB Test Cassette (Immunoassay)

CPT Code: 82274QW
Average Reimbursement: \$22.22

FOB Slide Test (Guaiac)

CPT Code: 82270 (Fecal Occult Blood, Diagnostic)
Medicare Code: G0107 (Fecal Occult Blood, Screening)
Average Reimbursement: \$3.50 (2000 National Cap)

To order, please contact your Authorized RediScreen distributor or call: (866) 737-6294