



Super Sani-Cloth® Germicidal Disposable Wipes



A Quaternary/High Alcohol Formula (55%) for the disinfection of hard, non porous surfaces and Medical equipment

- Bactericidal, tuberculocidal, and virucidal
- Kills TB and RSV in 1 minute; HBV, HCV, Human Coronavirus, Rotavirus, MRSA, VRE and ESBL Producing E. coli in 2 minutes
- Tested effective against 22 microorganisms
- Diamond-embossed wipe material is thick and strong
- Unique Deep Well Lid on canister seals securely to prevent moisture loss
- Available in convenient pocket packets

DESCRIPTION (SIZE)	REORDER NO.	PACKAGING
SUPER SANI-CLOTH® – LARGE WIPES (6" x 6.75")	Q55172	160/can, 12 cans/cs
SUPER SANI-CLOTH® – EXTRA-LARGE WIPES (8" x 14")	Q86984	65/can, 6 cans/cs
SUPER SANI-CLOTH® – LARGE IND. PACKETS (5" x 8")	H04082	50/box, 10 boxes/cs
SUPER SANI-CLOTH® – EXTRA-LARGE IND. PACKETS (11.5" x 11.75")	U87295	50/box, 3 boxes/cs

MATERIAL SAFETY DATA SHEET FOR SUPER SANI-CLOTH (M-935)
DISTRIBUTOR: [Mountainside Medical](#)

I-PRODUCT IDENTIFICATION

PRODUCT/TRADE NAME: **SUPER** SANICLOTH GERMICIDAL DISPOSABLE WIPE -U87295
HAZARD RATING (NFPA):
HEALTH: 1
FLAMMABILITY: 3 REACTIVITY: 0
SPECIFIC: NONE
EMERGENCY INFORMATION TELEPHONE NUMBER: CHEMTREC (800)424-9300 (24 HOUR)
CHEMICAL NAME: PROPRIETARY MIXTURE

II-HAZARDOUS INGREDIENTS PER 29 CFR 1910.1200

Hazardous Ingredients

%

ACGIH TLV

CAS NUMBER

ISOPROPYL ALCOHOL

55

400 ppm

67-63-0

QUATERNARY AMMONIUM CMPD

<1

N/A

27479-28-3

III-PHYSICAL/CHEMICAL CHARACTERISTICS

COLOR/ODOR/APPEARANCE: TOWELETTE SATURATED WITH CLEAR LIQUID WITH ALCOHOL ODOR

BOILING POINT: 181

°

F

FLASHPOINT: 74

°

F

VAPOR DENSITY: N/A

EVAPORATION RATE: N/A

SOLUBILITY IN WATER: COMPLETE

SPECIFIC GRAVITY (H

2

O = 1): 0.892

IV-FIRE & EXPLOSION HAZARD DATA

FLASH POINT (Method Used): 74

°

F TAG CLOSED CUP

LEL: N/A UEL: N/A

EXTINGUISHING MEDIA: WATER, FOAM

SPECIAL FIRE FIGHTING PROCEDURES: TREAT PRODUCT AS FLAMMABLE

UNUSUAL FIRE AND EXPLOSION HAZARDS: NONE

V-REACTIVITY DATA

STABILITY: STABLE

CONDITIONS TO AVOID: NONE

INCOMPATIBILITY: ANIONIC SURFACTANT, SOAPS

HAZARDOUS DECOMPOSITION OR BYPRODUCTS: NONE KNOWN

POLYMERIZATION: WILL NOT OCCUR CONDITIONS TO AVOID: NONE KNOWN

PRODUCT DOES NOT REACT WITH FORMALDEHYDE

VI-HEALTH HAZARD DATA

EFFECTS OF OVEREXPOSURE

SKIN: MAY CAUSE SKIN IRRITATION

EYES: MAY CAUSE IRRITATION

INHALATION: NOT NORMAL ROUTE OF ENTRY

INGESTION: NOT NORMAL ROUTE OF ENTRY

EMERGENCY AND FIRST AID PROCEDURES:

SKIN CONTACT: IF RASH OR IRRITATION DEVELOPS, DISCONTINUE USE

EYE CONTACT: FLUSH WITH COOL WATER FOR 15 MINUTES

INHALATION: REMOVE TO FRESH AIR

INGESTION: CONSULT PHYSICIAN

TARGET ORGANS: NONE KNOWN

KNOWN CARCINOGEN: NONE KNOWN

VII-SPILL AND DISPOSAL PROCEDURE

SPILL CONTROL: PICK UP WITH ABSORBENT WASTE

WASTE DISPOSAL METHOD: TREAT AS FLAMMABLE LIQUID. OBEY LOCAL, STATE AND FEDERAL REGULATIONS

HANDLING AND STORAGE: AVOID HIGH HEAT AND SOURCES OF IGNITION

VIII-CONTROL MEASURES/PROTECTION

RESPIRATION: USE NIOSH APPROVED RESPIRATOR WHEN HANDLING BULK LIQUID IF EXPOSURE LEVELS EXCEEDED

VENTILATION: RECOMMENDED

PROTECTIVE GLOVES: RECOMMENDED

EYE PROTECTION: IF SPLASHING MAY OCCUR

HYGIENIC PRACTICES: FOLLOW GOOD HOUSEKEEPING PRACTICES

OTHER: NONE

IX-TRANSPORT/SHIPPING

DOT SHIPPING NAME: CONSUMER COMMODITY

TECHNICAL SHIPPING NAME: N/A

DOT SHIPPING CLASSIFICATION: ORM-D

DOT ID NO.: N/A

DOT LABEL REQUIREMENTS: N/A

UN/NA NUMBER REGULATIONS: N/A

REPORTABLE QUANTITY: N/A

X-DISCLAIMER

THE INFORMATION FURNISHED HEREIN IS BELIEVED TO BE ACCURATE AND REPRESENTS THE BEST DATA CURRENTLY

Super Sani-Cloth® features and benefits

- **Unique Deep Well Lid** – Provides ample space to store and access next wipe
- **Diamond-embossed material** – Is thick and strong
- **Latex-free**
- **Pre-saturated with the correct amount of germicidal solution** – No mixing, no measuring – saves valuable time
- **Two (2) year shelf life**
- **EPA registered; meets CDC and OSHA guidelines** – Maintain compliance to protocols
- **Large and extra-large non-folded wipes in canisters; and folded wipes in individual packets** – More packaging and wipe size options for optimal placement and surface coverage
- **Sani-Bracket® Wall and Mobile Equipment Attachment System** – So versatile, place Super Sani-Cloth® germicidal disposable wipes on walls, IV poles, gurneys, anywhere they are needed most

Super Sani-Cloth® is an EPA registered hospital-grade disinfectant to accommodate the many situations and hard non-porous environmental surfaces found in healthcare settings.

Super Sani-Cloth® kills bacteria, viruses and yeast including:

TB (Mycobacterium bovis) • Burkholderia cepacia • Methicillin Resistant Staphylococcus aureus (MRSA)
Vancomycin Resistant Enterococcus faecalis (VRE) • Escherichia coli (E. coli)
ESBL Producing Escherichia coli (E. coli) • Pseudomonas aeruginosa • Salmonella enterica
Staphylococcus aureus • Klebsiella pneumoniae • Adenovirus Type 5 • Hepatitis B Virus (HBV)
Hepatitis C Virus (HCV) • HIV-1 Herpes Simplex Type 2 • Human Coronavirus
Influenza A2/Hong Kong • Respiratory Syncytial Virus (RSV) • Rhinovirus • Rotavirus • Vaccinia Virus • Candida albicans.

High Alcohol Formula (55%)

- **Bactericidal, tuberculocidal, and virucidal**
- **Tested effective against 22 microorganisms***
- **including TB, HBV and HCV**
- **Contact time is 2 minutes or less***

